



RE-ENTRY OPTIONS

In the event that emergent re-entry is required, the plate(s) can be removed by cutting the wire at the lateral edge of the plate. This can be achieved in the following ways:

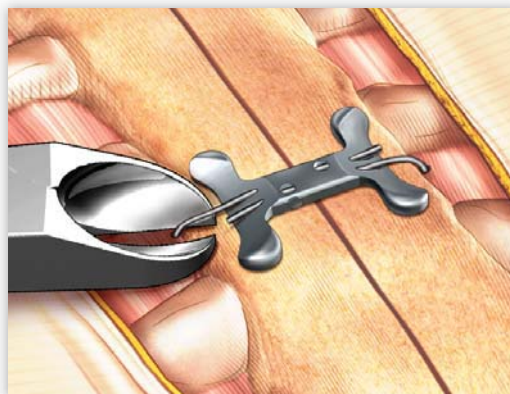

A. Use low profile wire cutters with a pointed tip to cut the wire at the ends of the plate.

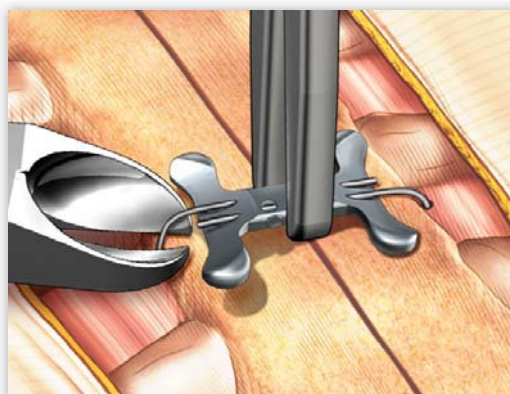

B. Use needle drivers to grab the plate and lift up to facilitate cutting the wire.



C. Wedge the cutter under the feet of the plate, or use an elevator, allowing access to cut the wire.



Warning: The plate is too strong to be cut with standard wire cutters. Only large orthopedic style plate cutters can be used to cut the plate.



21421 NW Jacobson Rd
Suite 700
Hillsboro, OR 97124

866.623.4137
www.acuteinnovations.com
customerservice@acuteinnovations.com

ACUTE will provide additional wire cutters upon request.